E-PAD

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Self-Adhering Neurostimulation Electrodes

Directions for use

Application:

1. Clean skin thoroughly. Electrodes do not stick well if any lotion, oil, make-up, dirt, etc. is left on skin.
2. Insert the pins of the leadwire from Praise and Punish into the electrode wire connectors so no metal is visible.
3. Remove electrodes from release liner and press firmly in place on skin. Adhesion improves when electrodes reach skin temperature.
4. If necessary, adjust stickiness by using water. To increase tack apply a small amount of water and air dry. To decrease tack and soften gel use more water.

Care and Storage:

1. Between uses return electrodes to the liner and store at room temperature.
2. The life of the electrodes varies depending on skin conditions, storage, amount of use, type of stimulation site. Usage may be extended by carefully cleaning the gel surface with water.
3. We suggest the adhesive pad should be changed to a new one after about ten times of use.
4. For personal use only.

Note:

DO NOT pull on the lead wires when removing electrodes from the release liner or skin. You may damage the electrodes if you do so.

Important:

Single patient use only. DO NOT apply to broken or injured skin. Should a skin rash occur, discontinue use and contact your physician.
DO NOT stimulate while driving a motorized vehicle or operating machinery.